

AUTO-WRAP®

Syringe Labeling System

Manufactured by Medical Packaging Inc.



DECREASE PACKAGING COSTS

INCREASE OPERATIONAL EFFICIENCY

REDUCE MEDICATION ERRORS



MPI's *Auto-Wrap® Syringe Labeling System* is an easy-to-use solution that automates the process of labeling syringes. This semi-automatic system produces a printed label and automatically wraps the printed label around the syringe in one single motion.



MPI's Auto-Wrap® Syringe Labeling System can accommodate any type of oral slip or injectable syringe from ½ mL up to 60 mL without any need for adjustment or changing of parts. The open face design and easy-to-use controls allow for easy loading of label stock and changeover.


Flagged and tacked labels are offered with print areas ranging from 1" – 1.5". Color flag labels can be used for various purposes, such as identifying pediatrics, variants/doses of a drug, storage conditions etc.

FLAGGED LABELS

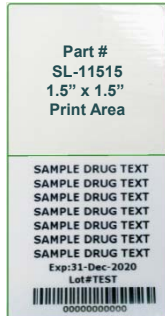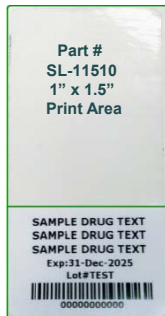

The top clear portion of the label is wrapped around the syringe with semi-permanent adhesive. The white/printed portion extends and hangs from the syringe. There is no adhesive behind the white area.

TACKED LABEL

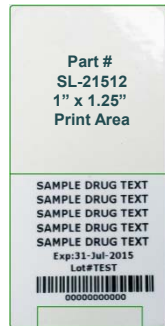

The top clear portion of the label is wrapped around the syringe with semi-permanent adhesive. The white/printed portion has a clear adhesive area which will keep the label tacked to the syringe. This will allow for better storage and for the label to be untacked from the syringe for viewing.

KEY FEATURES

- TIME SAVINGS** - An independent time motion study conducted on MPI's Auto-Wrap® concluded that time savings of 1 ½ - 2 ½ minutes per batch of 25 syringes can be achieved by implementing MPI's Auto-Wrap® Labeling System.

| Key Metrics | Syringe Size | Time Savings (per 25 Syringes) |
|-------------------|--------------|--------------------------------|
| Straight labeling | Any | 1 1/2 minutes |
| Folded labeling | 1 mL | 2 2/3 minutes |
| Folded labeling | > 5 mL | 2 1/3 minutes |

Study conducted by Summerfield Consulting

- COST SAVINGS** - Low cost per dose (CPD) . . . ~ 4 cents/label.
- PRODUCTIVITY** - Increase production throughput using automation while eliminating manual wrapping which can contribute to repetitive stress injuries (RSI).

To assure maximum adhesion and legibility, MPI's Auto-Wrap® Syringe Labeling System requires the use of label media developed by MPI in conjunction with MPI's *Pak-EDGE™ UD Barcode Labeling Software*, which is included with the labeling system.

TECHNICAL SPECIFICATIONS

| | |
|-------------|-----------------------------|
| Dimensions: | 17.5" W x 7.5" D x 11.0" H |
| Weight: | 25 lbs. |
| Safety: | CE, NRTL (MET) US, C |
| Power: | 115/230V, 50/60 Hz, 70 Watt |

The *Auto-Wrap® Syringe Labeling System* is the perfect complement to MPI's *Auto-Draw® Syringe Filling System!*

All MPI packaging systems are classified as US FDA 510(k) exempt and require the use of MPI authorized disposable materials.



8 Kings Court | Flemington | New Jersey | 08822 | USA
Toll-Free.800-257-5282 | Phone.609.466.8991 | Fax.609.466.3775 | www.medpak.com

DISTRIBUÉE PAR / DISTRIBUTED BY:

HEALTHMARK

1939 Onésime-Gagnon
Lachine, Québec, H8T 3M5, Canada

Sans Frais / Toll Free : 1-800-665-5492
Montréal : 514-336-0012
Fax : 514-336-7111



www.healthmark.ca