



2-Step Applicator Kit

For cleansing of contaminated surfaces

Towelette 1- High Potency Bleach

Contains 2% (W/W) specially formulated sodium hypochlorite soap solution. **Warning:** Keep out of reach of children. Do not get in eyes, on skin or on clothing. Can cause substantial but temporary eye injury-flush eyes with water. Avoid prolonged inhalation of vapors. Harmful if swallowed-call a physician.

Towelette 2- Bleach Inactivating Agent

Contains 1% (W/W) sodium thiosulfate solution and 0.9% (W/W) benzyl alcohol.
NOT FOR INTERNAL USE.

MADE IN USA

Manufactured for Acute Care Pharmaceuticals

Remember to wear latex or rubber gloves when using this product.

Pharma-Surface Guard™ packets need to be used in numerical order.

Step 1: Remove the double packet from the box and carefully tear open packet Number 1.

Step 2: Carefully remove the towelette from packet Number 1 and vigorously apply the pre-moistened towelette to the area to be treated.

Step 3: Allow the treated area to dry for at least 30 seconds.

Step 4: Remove the towelette from Packet Number 2 and apply pre-moistened towelette to the area previously treated with Packet Number 1.

Please note that streaking or a residue (salt residue) may appear after this application.

Step 5: If needed, the treated area may now be cleaned with an alcohol swab or simple soap solution (not provided) to resolve any inactive residue.

Step 6: Place the towelettes in a biohazard waste container.